INSTRUCTIONS FOR USE

EASY STORE
BREAST MILK STORAGE BAG

Package contents: 25 x Easy Store bags, Instructions for Use

Purpose:

The Easy Store breast milk storage bag is suitable for safe and compact storage and for easy heating of breast milk.

Important Information:

- The breast milk storage bag must be used once only.
- Thawed breast milk must not be refrozen.

Filling bag with breast milk:

- Wash your hands thoroughly with soap and water and dry them before using the bag.
- Label the green fields on the bag with a waterproof marking pen (name, date and time).
- Cut the top end of the bag along the dotted line and open the zip closure.
- Fill with breast milk to the 180 ml / 6 fl oz mark.
- Press the air out and close the zip closure.

Storage of breast milk:

- At room temperature: 3 to 4 hours is optimal; 6 to 8 hours under very clean conditions at 16–29 °C (60–85 °F)
- In the refrigerator: 72 hours is optimal; 5 to 8 days under very clean conditions at approx. 4 °C (39 °F)
- In the freezer: 6 months is optimal; 12 months is acceptable at <−17 °C (0 °F)

Heating breast milk:

- The bag is not suitable for thawing and heating in a microwave oven.
- Thaw breast milk gently in a warm water bath or in the refrigerator.
- Heat the bag in a warm water bath and feed the breast milk as soon it reaches body temperature.

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