Ardo natal perimassage

EN Instructions for use

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Ardo Natal Perimassage
Promotes the elasticity of the perineal tissue and provides optimal preparation for delivery.

Ardo Natal Perimassage was specially developed to increase the suppleness and elasticity of the skin in the perineal area. Almond oil, wheat germ oil and vitamin E nourish the skin and promote the elasticity of the perineal area.

Perineal massage: Perineal massage is used to increase the elasticity of the perineum and thus facilitates the natural birth process. Regular perineal massage during the final weeks before delivery can reduce the risk of perineal tears or episiotomy.

Application: Beginning in the 34th week of pregnancy, massage the perineal area at least 3-4 times a week for about 10-15 minutes with Ardo Natal Perimassage. Please wash your hands thoroughly before applying the product. Take an amount of the product that feels good to you on your fingers and start massaging the perineal area. Start with circular and stretching movements. If you have any questions, please do not hesitate to contact your doctor or midwife.

Shelf life and storage: Store in a cool, dry place away from light. Use within 12 months after opening.

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